

# PRESTAN Professional Adult Series 2000 Manikin

## Frequently Asked Questions



### **Can the PRESTAN Professional Adult Series 2000 Add-On Kit be used in a PRESTAN Professional Adult Manikin without monitor?**

Yes, the Series 2000 Add-On Kit can be used in a PRESTAN Professional Adult Manikin without monitor. Please refer to the PRESTAN Professional Adult Series 2000 Add-On Kit Installation Guide for upgrade instructions. **PLEASE NOTE:** The PRESTAN Adult Series 2000 Add-On Kit is only recommended for 2018 manikin models and newer.

### **Can the PRESTAN Professional Adult Series 2000 Add-On Kit be used in a PRESTAN Professional Adult Manikin with monitor?**

Yes, the Series 2000 Add-On Kit can be used in a PRESTAN Professional Adult Manikin with monitor. Please refer to the PRESTAN Professional Adult Series 2000 Add-On Kit Installation Guide for upgrade instructions. **PLEASE NOTE:** The PRESTAN Adult Series 2000 Add-On Kit is only recommended for 2018 manikin models and newer.

### **Can I install the PRESTAN Professional Adult Series 2000 Add-On Kit in the Jaw Thrust Manikins?**

Yes, the PRESTAN Professional Adult Series 2000 Add-On Kit can be installed in a PRESTAN Jaw Thrust Manikin. Please refer to the PRESTAN Professional Adult Series 2000 Add-On Kit Installation Guide for upgrade instructions. **PLEASE NOTE:** The PRESTAN Adult Series 2000 Add-On Kit is only recommended for 2018 manikin models and newer.

### **Can I install the PRESTAN Professional Adult Series 2000 Add-On Kit in the PRESTAN Professional Child or Infant Manikins?**

No, the PRESTAN Professional Adult Series 2000 Add-On Kit cannot be installed in the PRESTAN Professional Child or Infant Manikins.

### **How do you install the PRESTAN Professional Adult Series 2000 Add-On Kit?**

Please refer to the PRESTAN Professional Adult Series 2000 Add-On Kit Installation Guide and Installation Video.

### **How do you differentiate between the PRESTAN Professional Adult Manikin and the PRESTAN Professional Adult Series 2000 Manikin?**

The PRESTAN Professional Adult Manikin contains a blue CPR Monitor. The PRESTAN Professional Adult Series 2000 Manikin contains a medium skin tone CPR Monitor.

# PRESTAN Professional Adult Series 2000 Manikin

## Frequently Asked Questions



### **How do I install the PRESTAN CPR Feedback app?**

Please refer to the PRESTAN Professional Adult Series 2000 instruction manual and PRESTAN CPR Feedback app training videos for information on how to install the app.

### **How does the PRESTAN CPR Feedback app work?**

Please refer to the PRESTAN Professional Adult Series 2000 instruction manual and PRESTAN CPR Feedback app training videos for information on how the app works.

### **How do I connect my manikin to the PRESTAN CPR Feedback app?**

Please refer to the PRESTAN Professional Adult Series 2000 instruction manual and PRESTAN CPR Feedback app training videos for information on how to connect your manikin to the app.

### **What does the PRESTAN CPR Feedback app record?**

The PRESTAN CPR Feedback app records Rate, Depth, Recoil, Ventilations, Hands-off time and performance scores.

### **Does the PRESTAN Professional Adult Series 2000 Manikin meet the Heart & Stroke requirements for feedback?**

Yes, the PRESTAN Professional Adult Series 2000 Manikin along with the CPR Feedback app meets the Heart & Stroke requirements including rate, depth and recoil feedback, effective February 2021.

### **What devices are needed for the PRESTAN Professional Adult Series 2000 manikin and CPR Feedback app?**

For Apple devices: Compatible with iPhone, iPad and iPod touch. Requires iOS or iPadOS 13.0 or newer.

For Android devices: Requires Bluetooth® LE or 5.0 enable device running Android 5.0 or newer.

### **How many PRESTAN Professional Adult Series 2000 manikins can you connect to one device at a time?**

The PRESTAN CPR Feedback app can connect up to (6) PRESTAN Professional Adult Series 2000 manikins at one time.

### **What does the color coding of the scores in the CPR Feedback app mean?**

A score of greater than 70 is green, a score between less than 70 and greater than 10 is yellow and a score of less than 10 is red. Please refer to the PRESTAN Professional Adult Series 2000 instruction manual and PRESTAN CPR Feedback app training videos for information on how to interpret data from the app.

# PRESTAN Professional Adult Series 2000 Manikin

## Frequently Asked Questions



### **Does a high score mean I have passed a course and am certified?**

Although the PRESTAN CPR Feedback app records scores and provides a certificate of completion, the score is not based on any industry standards and it does not mean you are certified.

### **Can I use the PRESTAN Professional Adult Series 2000 manikin without app feedback?**

Yes, you can use the PRESTAN Professional Adult Series 2000 manikin without the app feedback. The LED's is the top left-hand shoulder of the manikin work exactly like the PRESTAN Professional Adult Manikins. The only difference is the audible feedback "beep" when compressions are performed with the PRESTAN Professional Adult Series 2000 manikin.

### **Are the PRESTAN Professional Adult Series 2000 lung bags the same as the original PRESTAN Professional Adult Manikin lung bags?**

No, the ventilation adult lung bags (VALB) have a breath sensor holder attached to the lung bag so that the breath sensor can be connected to the PRESTAN Professional Adult Series 2000 and ventilation feedback can be reported through the PRESTAN CPR Feedback App.

### **Can I use original (ALB) adult lung bags with this manikin?**

Yes, however the original adult lung bags (ALB) do not contain the breath sensor holder so you will not be able to record ventilation feedback through the PRESTAN CPR Feedback App with the ALB.

### **How do I insert and connect the PRESTAN Professional Adult Series 2000 ventilation lung bag?**

Please refer to the PRESTAN Professional Adult Series instruction manual for information on how to connect the ventilation lung bags.

### **What kind of batteries does the PRESTAN Professional Adult Series 2000 Add-On Kit require?**

The PRESTAN Professional Adult Series 2000 Add-On Kit requires (2)-AA batteries. **PLEASE NOTE:** When installing batteries into the CPR Monitor, make sure that they are installed correctly matching the positive (+) and negative (-) markings inside the battery compartment.

### **Do batteries come with this product?**

No, batteries do not come with this product.

### **What is the warranty on the PRESTAN Professional Adult Series 2000 manikin and Add-On Kit?**

PRESTAN carries a one-year warranty on the PRESTAN Professional Adult Series 2000 manikin and Add-On Kit.

# PRESTAN Professional Adult Series 2000 Manikin

## Frequently Asked Questions



### **Do I need to remove batteries after use?**

It is always best to remove the batteries after use. However, the PRESTAN Professional Adult Series 2000 manikin will go into a sleep mode after 15 minutes of no use.

### **How do I clean my PRESTAN Professional Adult Series 2000 Manikin?**

Please refer to the PRESTAN Professional Adult Series instruction manual.

### **Who do I contact for support if I am having issues with the PRESTAN Series 2000 Manikin or CPR Feedback app?**

Please contact your PRESTAN Authorized Distributor for support. If you forgot or do not have a distributor, then please contact PRESTAN to locate a distributor.